

Hornsby Comfy Hips®




Product Information

Proudly 100% Australian made, manufactured,
owned & operated using locally sourced materials.



Fitting Your Hornsby Comfy Hips®

It is important that the hip protectors fit properly. The diagrams below should assist you with this. If you draw an imaginary line from the top of your pubic hair line horizontally over to the top of the legs, this should go approximately through the centre of the shield.

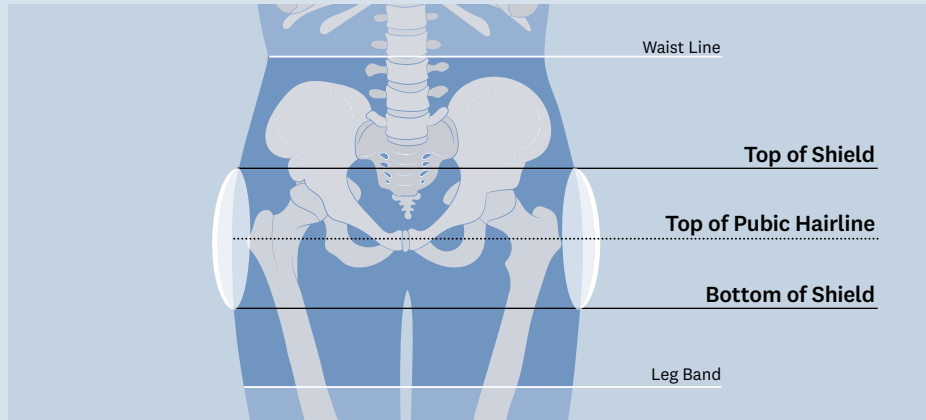


Correct Fitting of Shield

If the pants are fitting comfortably around the waist the shield will generally be in the correct position. If you have bought garments with removable shields, it does not matter which way up or which way around the shields are placed in the pockets. Protection is provided wherever dimples on shield cover the hip.

Wearing Your Hip Protectors

Because a fall cannot be predicted it is recommended that your hip protectors are worn at all times.

This means if you are getting up during the night they should be worn then also. Because they are made of soft silicone, these hip protectors are comfortable to wear to bed.

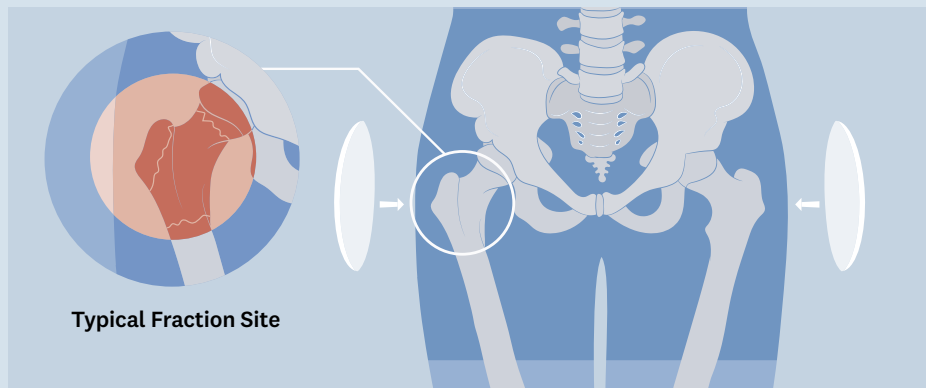

Caring for Your Product

Our underwear are machine washable in either cold or hot water. Trackpants are preferably washed in cold or warm water.

Garments will dry faster if shields are removed (in models that allow this). Some minor shrinkage may occur in cotton product after the initial wash.



Recommended washing at 70°C.



Do not wring.



Tumble dry, low heat.

Industrial washing processes tend to be harsher which may affect the life of your garment. For your peace of mind and safety, Hornsby Comfy Hips® shields are made of a soft dimpled silicone solid and are virtually indestructible.

Important Note

Hip protectors help to reduce the chance of breaking a hip in the event of a fall. They will not however prevent all hip fractures.

In the event of a car accident, a fall onto buttocks or severe cases of osteoporosis or other bone degenerating illness, a hip fracture may still occur.

Why choose Hornsby Comfy Hips®?

Integrity is at the core of our business. Our dedication to falls prevention and hip protection spans over 20 years.

We remain committed to ongoing research and development, and proudly support Australian materials and manufacturing processes.

- ✓ Registered manufacturer with Therapeutic Goods Administration
- ✓ Patented design
- ✓ We work closely with National Disability Insurance Scheme participants and providers
- ✓ Proudly 100% Australian made, manufactured, owned and operated using locally sourced materials
- ✓ Department of Veteran Affairs Gold and White Card holders can access our products at no charge*



Australian Government
Department of Health and Ageing
Therapeutic Goods Administration



Australian Government
Department of Veteran's Affairs



Australian Government
IP Australia



Our Commitment to Falls Protection

Staff of Hornsby Comfy Hips® have been associated with research, development, design, manufacture and marketing of hip protectors in Australia since the early 1990's. We continue to be associated with research, investigating ways of improving adherence with the wearing of hip protectors.

We believe we have specialised expertise and are able to offer our customers assistance with sound implementation or adherence issues advice. This product is registered with the Therapeutic Goods Administration of Australia.

Cross Matching Products

Hornsby Comfy Hips® pants and shields have been specifically designed as compatible items for maximum effectiveness. For this reason the cross matching of our shields or pants with other products is not endorsed by Hornsby Comfy Hips®



M 0497 829 851

T (02) 9653 3840

F (02) 9653 3845

hch@hornsbycomfyhips.com.au

Mail to: Hornsby Comfy Hips® Pty Ltd
PO Box 168 Galston NSW 2159 Australia



hornsbycomfyhips.com.au



**Any questions?
Please get in touch.**